Easy Plus II

Blood Glucose Test Strips

Easy Plus II Test Strip is for use with *Easy Plus II* or *Easy Talk* Glucose Meter of the same reference number only.

SUMMARY

The *Easy Plus II* Blood Glucose Monitoring System is intended for use in the quantitative measurement of glucose in fresh capillary whole blood from the finger and the forearm. It is intended for use by healthcare professionals and people with diabetes mellitus at home and as an aid in monitoring the effectiveness of a diabetes control program. The *Easy Plus II* Blood Glucose Monitoring System is not intended for the diagnosis of or screening for diabetes mellitus, nor for use with neonates.

TESTING PROCEDURE

Before you perform your first blood test, set up your meter correctly. You will need the meter, a test strip, and a lancing device with the lancet.

1. Wash your hands in warm, soapy water. Be sure to rinse and dry well.
2. With meter off, insert a new test strip with the contacts up into the test strip slot. The meter will turn on. You will see a full screen display briefly, and then a blinking blood drop icon will appear on the screen.
3. Place the cocked lancing device or single use disposable lancet against the side of the finger or forearm and press the trigger
4. Apply the blood drop to the front edge of the test strip when the blood drop icon is still blinking on the meter. Wait for 6 seconds, and the meter will display the test result.
5. Remove the used strip by hand or by pushing the ejector and the meter will power off with a display of "OFF" on the screen.

Alternate Site Testing (AST)

AST means that people use parts of the body other than the fingertip to check their blood glucose levels. *Easy Plus II* test strips provide you to test on the forearm with the equivalent results to fingertip testing. Please read the *Easy Plus II* user guide first before you do AST. When to use AST?

Medication, stress, illness, food and exercise can affect blood glucose levels. Capillary whole blood at the fingertip can reflect these changes faster than capillary blood at other sites of body. If you test your blood glucose level during or immediately after a meal, physical exercise or stressful events, take the blood sample from your fingertip instead of from other sites.

Use AST only at:

1. 2 hours or more after taking insulin;
2. 2 hours or more after a meal;
3. 2 hours or more after exercise.

Do not use AST if you are pregnant, or if you are aware of your glucose level is not stable as usual, or if you think you have hypoglycemia (low blood sugar) or hyperglycemia (high blood sugar). Do not use AST if you think your blood glucose is low and if your AST results do not match the way you feel.

CHEMICAL COMPOSITION

Glucose Oxidase (*Aspergillus niger*) 10.4%
Potassium Ferricyanide 53.1% Non-reactives 36.5%

PRECAUTIONS

-For *in vitro* diagnostic use only.

-*Easy Plus II* test strips can only be used with the *Easy Plus II* meter that has the same reference number.

The test strip will not work with any other meter and the meter should not be used with any other test strips.

-If this meter is used for multiple persons, caution should be taken to properly disinfect anything that comes in contact with blood or blood products.

-No changes in medication should be made based on the glucose results from this system unless recommended by your physician or healthcare professional.

-Incorrect high or low test results may have serious adverse medical consequences. See the User Guide for additional information.

STORAGE AND HANDLING

Check the expiration date on the test strip vial. Do not use beyond that date. Also, test strips should not be used longer than three (3) months after the vial is first opened. Store out of direct sunlight and store at temperatures of 8–30°C (46–86°F).

Once the test strip is removed from the vial, use immediately. Keep vial tightly capped, and do not reuse the test strips.

PRINCIPLE OF THE PROCEDURE

When glucose reacts with the reagents on the test strip, an electrical current is produced, which is proportional to the glucose concentration in the blood sample. The glucose concentration is calculated by the meter and based on the current measured.

ADDITIONAL MATERIALS REQUIRED

Easy Plus II Control Solution-Low or High. (The expected range of glucose for a Control Solution test is printed on the test strip vial.) Glucose Meter, Lancing Device and Sterile Lancets.

SPECIMEN COLLECTION AND PREPARATION

The preferred specimen for this system is capillary whole blood taken directly from a fingertip or forearm. For the best results, wash your hands in warm soapy water, rinse and dry before testing. Before testing, it is a common practice to wipe the fingertip with an alcohol swab. It is important to let the alcohol dry before puncturing the site.

QUALITY CONTROL

Easy Plus II Control Solution should be tested on *Easy Plus II* meter using *Easy Plus II* test strips to make sure the meter and test strips are working properly.

The Control Solution test should be run when a new vial of test strips is used.

Laboratories and hospitals should follow appropriate testing protocol regarding the quality control test.

Please see the User Guide on how to run a Control Solution test and make sure your test results are in the proper range printed on the test strip vial.

We suggest you should periodically compare the test system to another test system which is well maintained and monitored by a healthcare provider.

TEST RESULTS

Test results with *Easy Plus II* meter are plasma equivalent. No calculations of results are needed. Test results are displayed in either mg/dL or mmol/L, depending on how the meter has been set. Please see User Guide for more information and how to change the units.

Note: Your meter can show results either in "mg/dL" or "mmol/L" units. It is very important to use the correct unit of measurement to properly manage your diabetes.

If you live in the USA you should use mg/dL, your result will never have a decimal point. If you live in Canada and some European countries you should use mmol/L; your result will always have a decimal point. The mg/dL display is on by default.

LIMITATIONS OF THE PROCEDURE

1. Do not use for neonatal blood glucose testing.
2. Hematocrit in the range of 32–56% does not affect the glucose results. If you do not know your hematocrit level, consult your healthcare professional.
3. Hemoglobin levels of 500 mg/dL did not interfere with blood glucose test results.
4. Cholesterol levels up to 500mg/dL (12.9 mmol/L) and Triglycerides up to 1000 mg/dL has been shown not to affect glucose results.
5. Interference was observed for therapeutic level of L-Dopa. Uric acid, acetaminophen, ascorbic acid and ibuprofen have no significant interference in normal therapeutic level. However the concentration in blood beyond the levels below may cause incorrect results. Uric acid: 10.9 mg/dL; acetaminophen: 6.2 mg/dL; ascorbic acid: 4.5 mg/dL; ibuprofen: 37.5 mg/dL.
6. The test strips may be used at altitudes up to 5,280 feet (1,609m) without an effect on test results.
7. Persons suffering from severe dehydration should not be tested using a capillary whole blood sample.
8. Test results below 70 mg/dL indicate low blood glucose (hypoglycemia). Test results greater than 240 mg/dL indicate high blood glucose (hyperglycemia). If you get results below 70 mg/dL or above 240 mg/dL, repeat the test, and if the results are still below 70 mg/dL or above 240 mg/dL, consult your healthcare professional immediately!
9. Inaccurate results may occur in severely hypotensive individuals or patients in shock. Inaccurate results may occur for individuals experiencing a hyperglycemic-hyperosmolar state, with or without ketosis. Critically ill patients should not be tested with a blood glucose meter

EXPECTED VALUE REFERENCE²

Blood glucose levels normally will vary from time to time depending on food intake, medication dosages, health, stress or exercise. Consult your physician or healthcare professional for the target glucose value appropriate for you. Expected plasma blood glucose values for normal, nondiabetic adults are as follows.

Before eating	<100mg/dL
Two hours after meals	<140mg/dL

If you are experiencing symptoms that are not consistent with your blood glucose results and you have followed all instructions in this user guide, call your healthcare professional. Never ignore symptoms or make significant changes to your diabetes control programs without speaking to your healthcare professional.

PERFORMANCE CHARACTERISTICS

For Consumers :

Accuracy: Diabetes experts have suggested that glucose meters should agree within 15mg/dL of a laboratory method when the glucose concentration is lower than 75 mg/dL, and within 20% of a laboratory method when the glucose concentration is 75 mg/dL or higher. The chart below displays how often the *Easy Plus II* meter achieves this goal. The chart is based on a study done on 159 samples from 159 patients to see how well the *Easy Plus II* compared to laboratory results.

1. Fingertip

For glucose results lower than 75 mg/dL, the percent (and number) of meter results that match the laboratory method within 15 mg/dL :	100% (19/19)
For glucose results at 75 mg/dL or higher, the percent (and number) of meter results that match the laboratory method within 20%:	100% (140/140)

2. Forearm

For glucose results lower than 75 mg/dL, the percent (and number) of meter results that match the laboratory method within 15 mg/dL :	100% (19/19)
For glucose results at 75 mg/dL or higher, the percent (and number) of meter results that match the laboratory method within 20%:	100% (140/140)

Note: When meter results are compared to the laboratory results, results below 75 mg/dL are compared in mg/dL.

Additional Information for Healthcare Professionals

This system is evaluated according to ISO 15197. Measurement range: The measurement range of the *Easy Plus II* Glucose Monitoring System is 20 to 600 mg/dL.

Easy Plus II meter gives plasma equivalent results.

Fig. 1 Finger plot


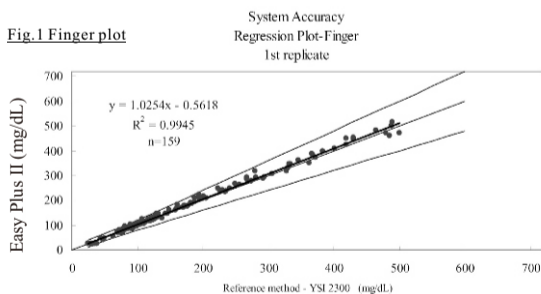
Fig. 2 Forearm plot

REFERENCES

1. American Diabetes Association-Diabetes Forecast (website information <http://www.forecast.diabetes.org/>)
[http://www.forecast.diabetes.org/diabetes-101/hyperglycemia.](http://www.forecast.diabetes.org/diabetes-101/hyperglycemia) [http://www.forecast.diabetes.org/diabetes-101/hypoglycemia.](http://www.forecast.diabetes.org/diabetes-101/hypoglycemia))
2. American Diabetes Association: Diabetes Care, January 2007, volume 30 (Suppl. 1) S42-S47.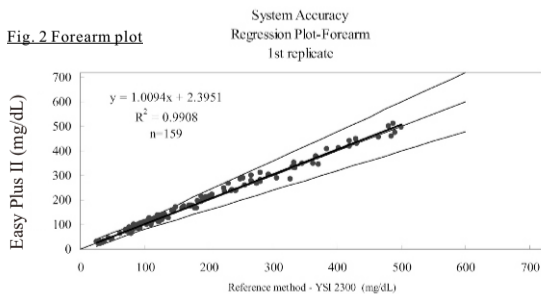

Distributed by:
Home Aide Diagnostics, Inc.
1072 S. Powerline Rd., Deerfield Beach, FL 33442, USA
TEL: 1-800-915-0116 (9 AM to 9 PM EST/ Mon. to Fri.
Outside these times, contact your healthcare professional)

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