Easy Plus II Control Solution is for use with Easy Plus II or Easy Talk Glucose Meter.

SUMMARY AND INTENDED USE

Easy Plus II Glucose Control Solution is intended for Quality Control of Easy Plus II Glucose test system. The Control Solution helps to ensure that the system is working correctly. Always run a Control Solution test whenever you open a new vial of test strip.

Easy Plus II Control Solution contains a measured amount of glucose that reacts with Easy Plus II Test Strips. A test with Control Solution is similar to a blood test except that you use Easy Plus II Control Solution instead of a drop of blood. The test result using the Control Solution should fall within an expected range of results, which you will find printed on the test strip vial that you are using.

CHEMICAL COMPOSITION

D-glucose, Viscosity Modifier, Preservatives

PRECAUTIONS

1. Easy Plus II Glucose Control Solution is for **in vitro** diagnostic use only.
2. Easy Plus II Glucose Control Solution is made for Easy Plus II Glucose Monitoring system use only.
3. Do not use any other control solutions.
4. Check the expiration date on the bottle. Do not use if expired.
5. Shake the bottle gently, discard the first drop of Easy Plus II Control Solution, and wipe off the dispenser tip to ensure a good sample and an accurate result.
6. Use only for 3 months after first opening. Record the open date on the Control Solution box. Discard after 3 months.

STORAGE AND HANDLING

Easy Plus II Glucose Control Solution should not be used longer than 3 months after the bottle is first opened. Store at 46-86°F (8-30°C). Keep the Control Solution bottle tightly capped when not in use.

PRINCIPLE OF THE PROCEDURE

When glucose in the Control Solution reacts with the reagents on the test strip, an electrical current is produced which is proportional to the glucose concentration in the Control Solution. The glucose concentration is calculated by the meter, based on the current measured.

MATERIALS SUPPLIED

1 bottle of Easy Plus II Glucose Control Solution-Low or High, 3mL. (The expected range of glucose for a Control Solution test is printed on the test strip vial.)

ADDITIONAL MATERIALS NEEDED

Easy Plus II Glucose Meter
Easy Plus II Glucose Test strips

TESTING PROCEDURE

To perform a glucose test with Easy Plus II Glucose Control Solution, follow the five steps below:

1. Insert a strip into the meter, a full LCD screen will be displayed briefly and then a blood drop icon appears.
2. Press the C key, and the meter will display a letter "C" on the upper right corner of screen, indicating the control test mode. The test result will not be stored in the memory.
3. Squeeze a small amount of Control Solution on a flat surface and apply to the front edge of the test strip when the blood drop icon is blinking on the meter.
4. Wait for 6 seconds, and the meter will display the result.
5. Remove the used strips by hand or by pushing the ejector and the meter will power off with a display of "OFF" on the screen.

Note: Always make sure you press C key for Control Solution test, otherwise the control solution test result will be memorized and affect your "AVG" results.

EXPECTED CONTROL RESULTS

The test result using the Control Solution should fall within an expected range printed on the test strip vial. If your results fall outside the expected range, repeat the test. Results that repeatedly fall outside the expected range may indicate:

- *You may not be doing the test correctly; repeat the test, carefully following instructions.
- *Your meter is not coded for the strips you are using.
- *The control solution is expired or contaminated.
- *The test strips are damaged or outdated.
- *There is a problem with the meter.

IMPORTANT:

If you continue to get Control Solution test results that fall outside of the expected range repeatedly, **DO NOT use the system to test your blood.** Contact your healthcare provider for medical questions. Contact Customer Service for problems with the glucose monitoring system.

AVAILABILITY

To order supplies, or obtain additional control solution levels, please contact your local distributor as below.

Home Aide Diagnostics, Inc.
1072 S. Powerline Rd. Deerfield Beach, FL 33442, USA
TEL: 1-800-915-0116 (9 AM to 9 PM EST/Mon. to Fri.
Outside these times, contact your healthcare professional)

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