

INSTRUCTIONS MANUAL FOR SAFE USE

IMPORTANT SAFETY INSTRUCTIONS DO NOT DESTROY

WARNING: To reduce burning risk, electric shock, fire and bodily injuries, this product must be used in accordance with the following instructions.

1. Read all instructions.
2. Do not use while sleeping.
3. Do not use on infants or animals.
4. This pad is not to be used by or on an invalid, a sleeping or unconscious person, or a person with poor blood circulation unless prescribed by a doctor.
5. Do not use on sensitive skin.
6. May cause burning. Check the skin frequently.
7. Do not use the pad without the cloth cover.
8. Do not use in an oxygen-enriched environment or near equipment, which stores or emits oxygen.
9. Do not use pins or metallic fasteners on this pad.
10. Do not sit on or lay on top of the heating pad.
11. Do not crush it.
12. Avoid sharp folds.
13. Do not pull the electric cord.
14. Do not use the cord as a handle.
15. Examine the inner cover of the pad before each use; discard the pad if inner cover shows any signs of deterioration. If damaged please discard the pad.
16. Do not use the heating pad on children without adult supervision.
17. Use the heating pad in 110-120 volt electric outlet.
18. Do not try to break the heating pad.
19. No parts on heating pad are replaceable. If for any reason the heating pad is not working see warranty and customer services information.
20. Do not use the heating pad with any lotion or gels that create heat. This may burn the skin.
21. Place the heating pad on top not under the part of the body that needs heat.

HOW TO USE YOUR HEATING PAD FOR DRY HEAT

WARNING: Never use this heating pad without the cloth cover.

1. Stretch the cloth cover over a flat surface with the printed side facing up.
2. Insert the vinyl pad into the cover, making sure the control and cord are outside the cover.
3. Close the cover with the snap button.
4. Plug it into an electrical outlet.
5. To turn the heating pad ON, press the button. Your heating pad WILL NOT come on until you press the button.
NOTE: If the heating pad still does not work try plugging in it into another electrical outlet. Make sure the electrical cord is not damaged or broken.
6. Let the pad preheat for a few minutes.
Your heating pad has been designed to work with the maximum heating according to UL standards.
7. Select the desired temperature setting: Press the button, selecting WARM, LOW, MEDIUM, or HIGH.
The heating pad will turn off after cycling through the heat settings.
The heating pad will automatically turn off after 40 minutes.

**NOT FOLLOWING THESE INSTRUCTIONS
WILL VOID THE WARRANTY IMMEDIATELY.**

USE OF THE MOIST HEAT THERAPY

WARNING: Do not immerse the vinyl heating pad in water or place it under running water. This can cause severe damage.

1. Remove the sponge from the heating pad. Place the sponge under warm running water until it is fully saturated. Gently wring out the excess water.
2. Place the sponge on top of the pad and cover both with the cloth cover. Close the cover with the snap button.

CARE INSTRUCTIONS

Remove cloth cover over the heating pad. To keep the cover fresh and clean, wash it with mild soap in lukewarm water and drip dry to keep it soft.
DO NOT MACHINEWASH OR DRY.

It is to be expected that the cover of the heating pad unit may eventually become discolored. Excessive scorching may cause the cover to deteriorate.

SAVE THESE INSTRUCTIONS

For products purchased in the United States and Canada only: to reduce the risk of electric shock, this product contains a polarized plug (one prong is thicker than the other). This plug fits in a one-way polarized plug outlet only. If the plug does not fit in the electrical outlet, turn the plug around. If it still doesn't fit call an electrician.
DO NOT TRY TO DEFEAT THIS SECURITY ISSUE.