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HOW TO USE YOUR COMFORT HEAL AUTO-OFF HEATING PAD

For models with a removable cloth cover, lay cloth cover on a flat surface with the printing face up. Insert the vinyl pad into the cover, making sure the cord and the control remain on the outside of the cover. Snap the cover closed.

WARNING: Never use this type of pad without the cloth cover.

1. Plug supply cord into 120V AC electrical outlet.
2. To turn pad on, press Push-Button. The heating pad will NOT turn on until you press the Push-Button.
NOTE: If the heat level indicators do not light up or stay on, the plug of control connector may not be firmly or completely connected. Unplug cord from wall outlet, check the pad connection and plug safety cord into outlet again.
3. Allow pad to preheat for several minutes. The heating pad has been designed to meet the maximum heat output according to UL standards.
4. Select from 4 heat settings by pressing Push-Button successively to achieve desired heat level.
5. To turn pad OFF, you must scroll through the remaining heat setting by pressing the Push-Button sequentially.

Using Moist Heat Option

The heating pad comes with a separate anti-microbial sponge for moist heat.

- Run warm water over sponge until it is thoroughly saturated.
- Gently wring out excess water.
- Place the sponge on top of the heating pad and slip the cloth over the heating pad. Snap cover closed.
- Plug into a 110-120 Volt AC electrical outlet
- Set control to the desired heat.
- Store the anti-microbial sponge in a cool, dark, place after each use.

Fabric Cover Washing Instruction

- Disconnect pad from outlet.
- Remove cover from pad
- Machine or hand wash; use mild detergent. Wash with cool water only.
- Hang to dry or machine dry on gentle, cool cycle.

IMPORTANT SAFETY INSTRUCTIONS
DO NOT DESTROY THIS MANUAL

DANGER: TO REDUCE THE RISKS OF BURNS, ELECTRIC SHOCK, FIRE, AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS.

1. Read all instructions
2. Do not use while sleeping.
3. Do not use on an infant or an animal.
4. This pad is not to be used on or by an invalid, an unconscious person or a person with poor blood circulation.
5. Do not use on areas of insensitive skin.
6. Check skin under pad frequently. Burns can occur regardless of control setting.
7. Never use pad without cover in place.
8. Do not use in oxygen enriched environment.
9. Do not use pins or other metallic means to fasten pad in place.
10. Do not sit on or crush pad. Avoid shape folds while using pad.
11. Never pull this pad by the supply cord.
12. Do not use the cord as a handle.
13. Carefully examine inner cover before each use. Discard the pad if inner covering shows any signs of deterioration.
14. This unit should not be used by children without adult supervision.
15. Use this pad only on a 110-120 volt ac circuit. Unplug when not in use.
16. Do not tamper with this pad in any way.
17. There are no user-serviceable parts. If for any reason this pad does not function satisfactorily, see warranty for consumer service information.
18. Do not use this pad with liniment, salve, or ointment preparations, especially ones that contain heat-producing ingredients. Skin burns could result.
19. Place pad on top of part of body needing heat.
20. Save these instructions.

TO REDUCE THE RISK OF ELECTRICAL SHOCK, THIS APPLIANCE HAS A POLARIZED PLUG (ONE BLADE IS WIDER THAN THE OTHER). THIS PLUS FITS POLARIZED OUTLET ONLY ONE WAY. IF THE PLUG DOES NOT FIT FULLY INTO THE OUTLET, REVERSE THE PLUG. IF IT STILL DOES NOT FIT, CONTACT A QUALIFIED ELECTRICIAN.

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