


Pro Comfort Foot Bath

Hydrotherapy System

MODEL: PWFM-1

READ ALL SAFETY INSTRUCTIONS PRIOR TO USE

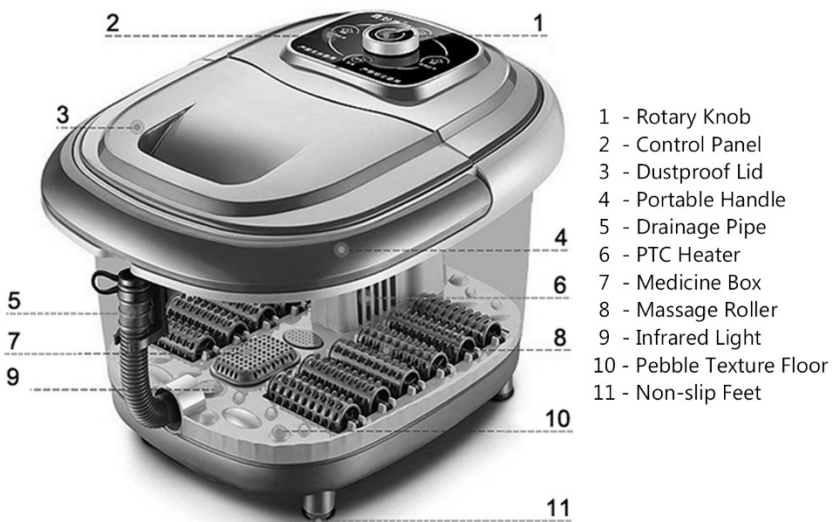

Caution: To avoid injury and irreparable damage, always remain seated when using this footbath. Never stand on or in the unit.

CAUTION: THIS UNIT CAN ONLY RUN WITH WATER IN IT. DO NOT USE WITHOUT WATER.

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTION SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for a product that has fallen into water. Unplug immediately.
- DO NOT place or store appliance where it can fall or be pulled into tub or sink.
- DO NOT place in or drop into water or other liquid.

WARNING – TO REDUCE THE RISK OF BURNS, FIRE, ELECTRIC SHOCK OR INJURY TO PERSONS:

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, or if it has been dropped or damaged or dropped into water. Return the appliance to Home Aide Diagnostics, Inc. along with your receipt or proof of purchase for replacement.
- Keep the cord away from heated surfaces.
- DO NOT handle plug with wet hands.
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- NEVER operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock or injury to a person.
- DO NOT carry this appliance by supply cord or use cord as a handle.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair and the like.
- NEVER drop or insert any object into any opening.
- DO NOT use outdoors.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by Home Aide Diagnostics, Inc.
- DO NOT stand on or in the appliance, Use only while seated.
- Use heated surface carefully. May cause serious burns. DO NOT use over insensitve skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

IF YOU HAVE OR HAVE HAD ANY SKIN CONDITIONS, DISEASES AND/OR HEALTH PROBLEMS, YOU SHOULD CONSULT YOUR PHYSICIAN BEFORE USE.

- Consult your doctor prior to using this product if:
 - You are pregnant
 - You have a pacemaker
 - You are diabetic
 - You have any concerns regarding your health
- DO NOT use while bathing or in the shower. Use only in a dry area
- DO NOT use on textiles with wired heads such as towels and carpets in order to prevent machine failure caused by thread head wool entering the appliance
- DO NOT attempt to plug or unplug unit while feet are in the water
- Use appliance on a sturdy or level surface
- We recommend using this unit for no longer than 30 minutes at a time. Extensive use could cause the product to overheat. Should this occur, discontinue use and allow the unit to cool before operating again.
- If you experience pain in a muscle or joint for prolonged period of time, discontinue use and consult your doctor. Persistent pain could be a symptom of a more serious condition.
- NEVER use any footbath on open wounds, discolored areas, or any area of the body that is swollen, burned, inflamed or where skin eruptions or sores are present.
- Use of this product should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.
- NEVER use while sleepy or drowsy.
- Always unplug the unit when filling or moving the unit.
- Ensure that your hands are dry when operating the controls or removing the plug.
- This product should NEVER be used by an individual suffering from any ailments that would limit the user's capacity to operate the controls.
- This product should NEVER be used by any individual suffering from any ailments that would limit or altogether inhibit the user's capacity to feel or have any sensation in the body part that the product is being used for.
- This product is intended for household use only.

SAVE THESE INSTRUCTIONS

OPERATING INSTRUCTIONS

1. Ensure that the unit is unplugged before filling with water. Fill with warm water to the fill line located on the inside of the tub.
CAUTION: DO NOT FILL WATER OVER THE MAX FILL LINE
2. Plug the unit into a 120 volt household outlet.
3. Turn the foot bath on by moving the knob on the control panel to one of the functions. You may choose HEAT, BUBBLES or HEAT & BUBBLES.
4. While seated, place your feet in the footbath.
5. Enjoy a deep soothing massage by sliding feet back and forth over the raised nodes on the foot bath floor.
6. After use, turn the unit off by turning the knob to on/off. Unplug from the outlet and pour the water out using the drainage pipe AFTER the device has cooled down. Don't pour the water over the control panel.

MAINTENANCE

TO CLEAN - Unplug the unit and allow it to cool before cleaning. Clean only with a soft, damp cloth and mild detergent. Use the Drainage Pipe to pour out the water from the foot bath. Release the pipe from the holder and the water will drain out.

- After cleaning, wipe surfaces with a dry cloth
- Never use strong abrasives, detergents or cleansing agents
- Periodically straighten cord if it becomes twisted

TO STORE

Unplug the appliance from the outlet and allow it to cool and dry

- Store in its box or a clean, dry place
- To avoid breakage, do not wrap the power cord around the unit
- Do not hang the unit by the power cord.
- This appliance is designed for household use only.

TWO YEAR LIMITED WARRANTY. For full warranty details

Visit <https://homeaide.us/warranty.html>

FOR CUSTOMER SERVICE:

E-mail: info@homeaide.us

Call: 1-800-915-0116 Monday-Friday 9:00am-5:00pm EST

Para el Manual en Español, llame al 1-800-915-0116 o vaya a <https://homeaide.us/products.html> y seleccione "Comfort Aides" y en esa sección seleccione "Pro Comfort Foot Bath"